

## **Instructions for Use Manual**

### GeniCure Genital Herpes Treatment System

Thank you for choosing the GeniCure system! This guide will show you how to use it to safely and effectively manage herpes outbreaks.

---

### **What's in the Box?**

1. Samaritan CS Polymer: A special polymer that protects and helps heal sores.
2. GeniCure Device: A light therapy tool designed to speed up healing.

---

### **Important Reminders**

- Always follow this guide to use the GeniCure system correctly.
- Do not use the device on healthy skin or areas without sores.
- Consult a doctor if you are pregnant, breastfeeding, or have other medical conditions.
- Keep the device and polymer out of reach of children.
- Do not share the device or polymer to avoid spreading germs.


### **How to Use GeniCure**

#### Step 1: Prepare the Affected Area

1. Wash your hands thoroughly with soap and water.
2. Gently clean the sore with warm water and pat it dry using a clean, soft towel.

#### **Step 2: Apply the Samaritan CS Polymer**

1. Open the syringe of Samaritan CS Polymer.
2. Squeeze a entire amount at once onto the sore, ensuring the entire sore is covered.

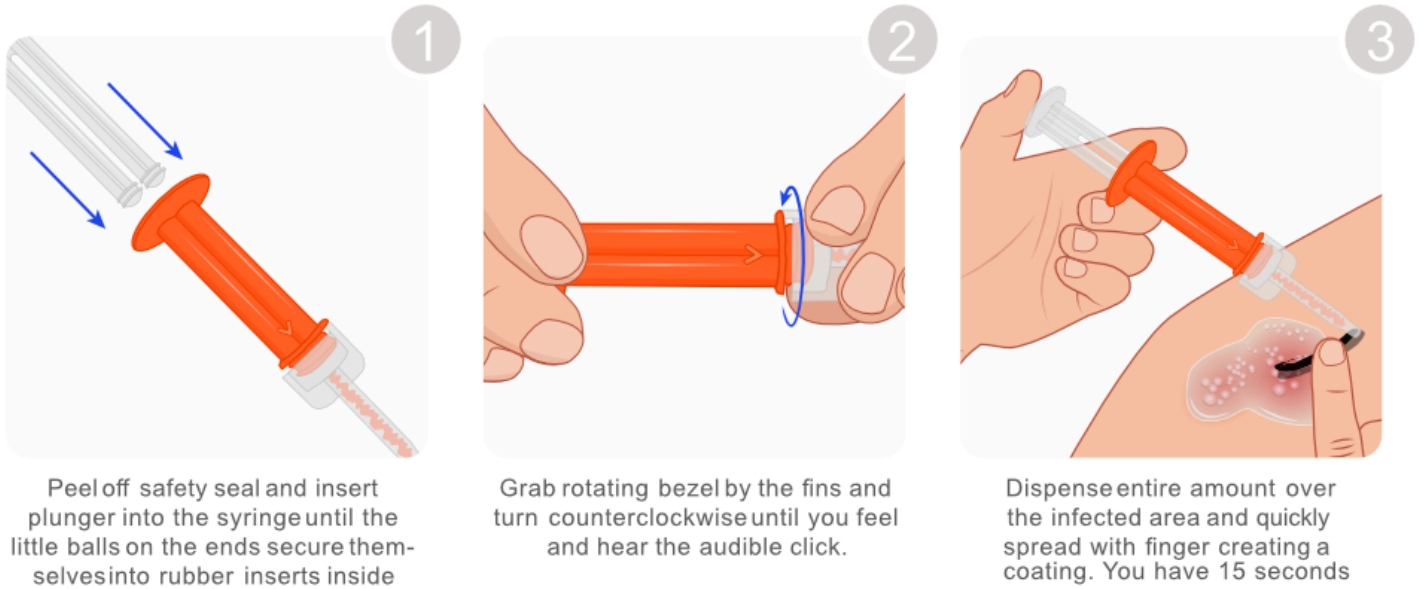

3. Spread the polymer evenly using a clean finger or applicator within 15 seconds


4. Wait 1-2 minutes for the polymer to form a protective layer.

### Step 3: Set Up the GeniCure Device

1. Remove the GeniCure device from its box and ensure it is fully charged or equipped with fresh batteries.
2. Press the power button to turn the device on. A light will indicate that it is ready for use.
3. The device is pre-programmed, so no adjustments are needed.

### Step 4: Treat the Sore

1. Place the GeniCure device gently over the sore, **ensuring contact** with the polymer-covered area.
2. Hold the device steady until it automatically shuts off, signaling the session is complete.

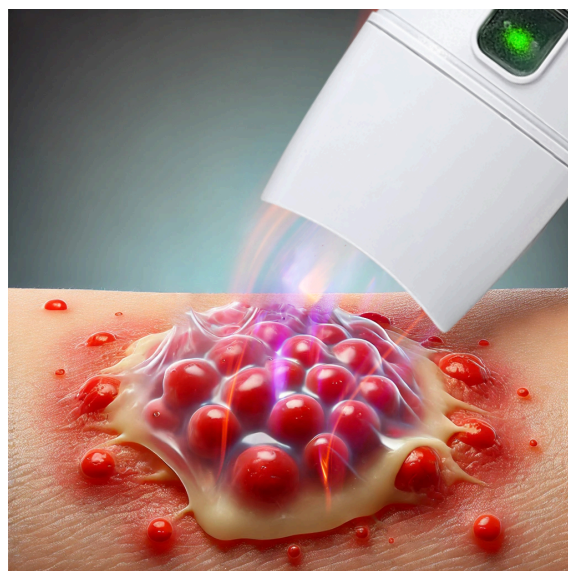

## **Step 5: After Treatment**

1. Carefully remove the device from the sore.
2. Leave the polymer on the sore to maintain its protective barrier.
3. Clean the treatment head of the device with isopropyl alcohol.
4. Store the device and polymer in a clean, dry place.

## **When to Use GeniCure**

- Begin treatment as soon as you notice signs of an outbreak.
- Use the device up to three times a day, with at least 4 hours between sessions.

## **Caring for Your GeniCure Device**

1. Clean the device with a soft, damp cloth after each use.
2. Avoid submerging the device in water or other liquids.
3. Store the device and polymer in a cool, dry place away from direct sunlight.

## **Warranty and Support**

Your GeniCure device comes with a 1-year warranty covering manufacturing defects. For assistance, please contact our customer support team at [Customer Support Contact Information].

## **Things to Remember**

- Stop using the system and consult a doctor if you experience irritation or an adverse reaction.
- GeniCure is not a cure for herpes but is designed to help manage outbreaks and alleviate symptoms.